
## Olympic LED Exam Light

Model #50970

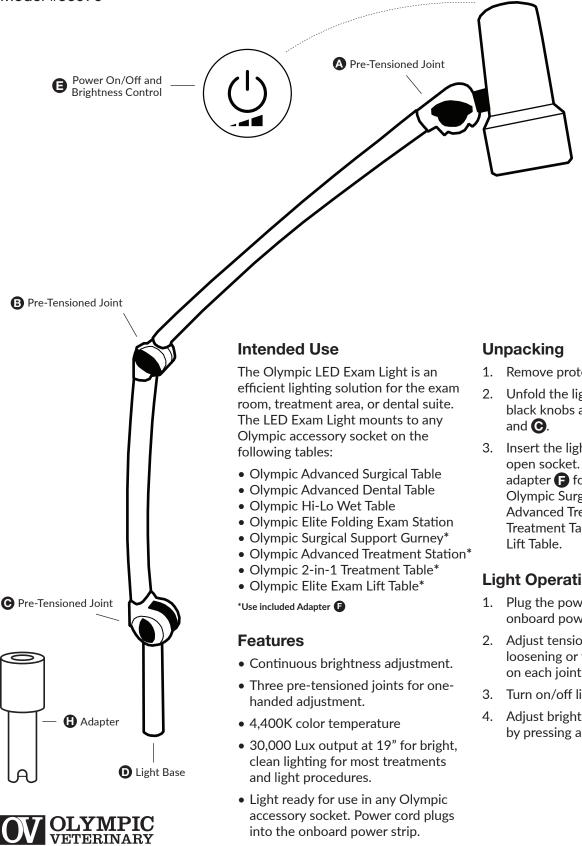

- 1. Remove protective wrapping.
- Unfold the light by loosening the black knobs at each joint **A**, **B**,
- 3. Insert the light base **D** into any open socket. Use the included adapter **(**) for inserting into the Olympic Surgical Support Gurney, Advanced Treatment Station, 2-in-1 Treatment Table, and the Elite Exam

## Light Operation

- 1. Plug the power cord into your table's onboard power strip.
- 2. Adjust tension on the light by loosening or tightening the knobs on each joint **(A)**, **(B)**, and **(C)**.
- 3. Turn on/off light by pressing  $\square$ .
- 4. Adjust brightness to your preference by pressing and holding  $\boldsymbol{\Theta}$ .