


# Olympic Advanced LED Procedure Light

Model #50971

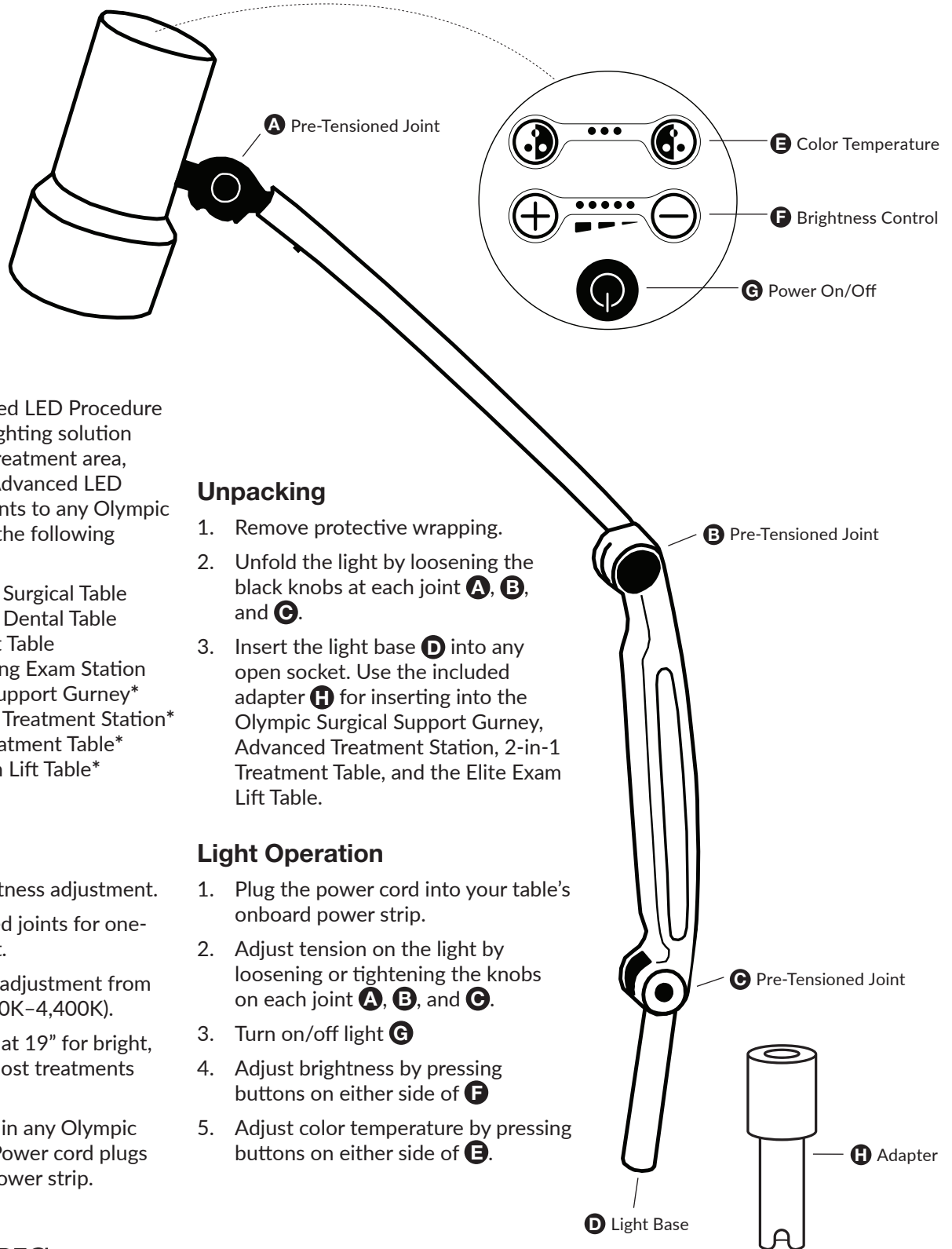

## Intended Use

The Olympic Advanced LED Procedure Light is an efficient lighting solution for the exam room, treatment area, or dental suite. The Advanced LED Procedure Light mounts to any Olympic accessory socket on the following tables:

- Olympic Advanced Surgical Table
- Olympic Advanced Dental Table
- Olympic Hi-Lo Wet Table
- Olympic Elite Folding Exam Station
- Olympic Surgical Support Gurney\*
- Olympic Advanced Treatment Station\*
- Olympic 2-in-1 Treatment Table\*
- Olympic Elite Exam Lift Table\*

\*Use included Adapter **H**

## Features

- Five levels of brightness adjustment.
- Three pre-tensioned joints for one-handed adjustment.
- Color temperature adjustment from warm to cool (3,300K-4,400K).
- 50,000 Lux output at 19" for bright, clean lighting for most treatments and procedures.
- Light ready for use in any Olympic accessory socket. Power cord plugs into the onboard power strip.

## Unpacking

1. Remove protective wrapping.
2. Unfold the light by loosening the black knobs at each joint **A**, **B**, and **C**.
3. Insert the light base **D** into any open socket. Use the included adapter **H** for inserting into the Olympic Surgical Support Gurney, Advanced Treatment Station, 2-in-1 Treatment Table, and the Elite Exam Lift Table.

## Light Operation

1. Plug the power cord into your table's onboard power strip.
2. Adjust tension on the light by loosening or tightening the knobs on each joint **A**, **B**, and **C**.
3. Turn on/off light **G**
4. Adjust brightness by pressing buttons on either side of **F**
5. Adjust color temperature by pressing buttons on either side of **E**.



Need help setting up your table?  
Schedule a video call today: [www.OlyVet.com/Showroom](http://www.OlyVet.com/Showroom)

1-800-789-6867  
[www.olympicveterinary.com](http://www.olympicveterinary.com)