


Olympic LED Exam Light

Model #50970

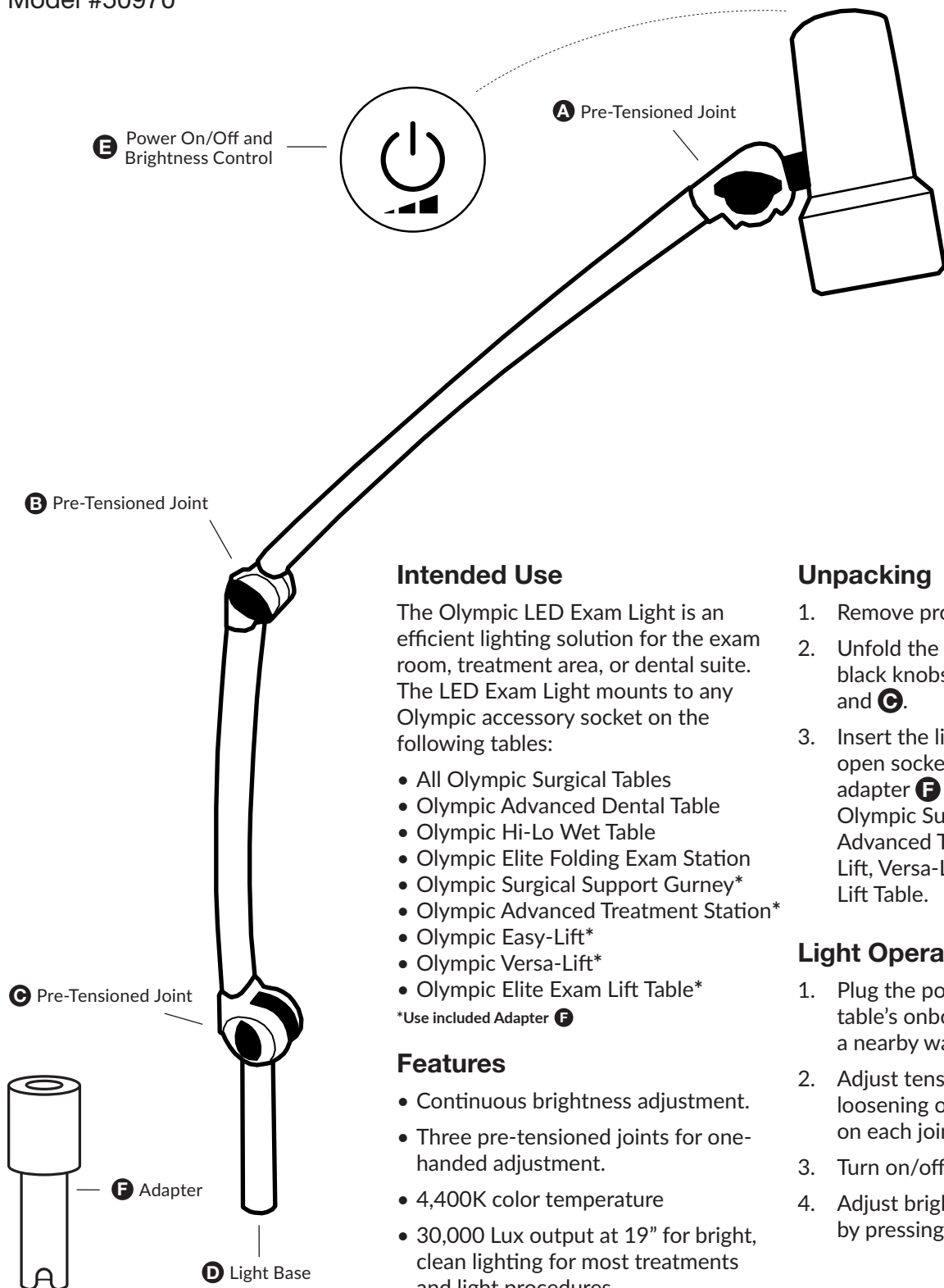

Intended Use

The Olympic LED Exam Light is an efficient lighting solution for the exam room, treatment area, or dental suite. The LED Exam Light mounts to any Olympic accessory socket on the following tables:

- All Olympic Surgical Tables
- Olympic Advanced Dental Table
- Olympic Hi-Lo Wet Table
- Olympic Elite Folding Exam Station
- Olympic Surgical Support Gurney*
- Olympic Advanced Treatment Station*
- Olympic Easy-Lift*
- Olympic Versa-Lift*
- Olympic Elite Exam Lift Table*

*Use included Adapter **F**

Features

- Continuous brightness adjustment.
- Three pre-tensioned joints for one-handed adjustment.
- 4,400K color temperature
- 30,000 Lux output at 19" for bright, clean lighting for most treatments and light procedures.
- Light ready for use in any Olympic accessory socket. Power cord plugs into the onboard power strip.

Unpacking

1. Remove protective wrapping.
2. Unfold the light by loosening the black knobs at each joint **A**, **B**, and **C**.
3. Insert the light base **D** into any open socket. Use the included adapter **F** for inserting into the Olympic Surgical Support Gurney, Advanced Treatment Station, Easy-Lift, Versa-Lift, and the Elite Exam Lift Table.

Light Operation

1. Plug the power cord into your table's onboard power strip or a nearby wall socket.
2. Adjust tension on the light by loosening or tightening the knobs on each joint **A**, **B**, and **C**.
3. Turn on/off light by pressing **E**.
4. Adjust brightness to your preference by pressing and holding **E**.



Need help setting up your table?

Schedule a video call today: www.OlyVet.com/Showroom

1-800-789-6867

www.olympicveterinary.com

AM_50970_0819Q