


Operator's Manual


Olympic Elite Folding Exam Station™

Model 52001, 52002

Serial #: _____

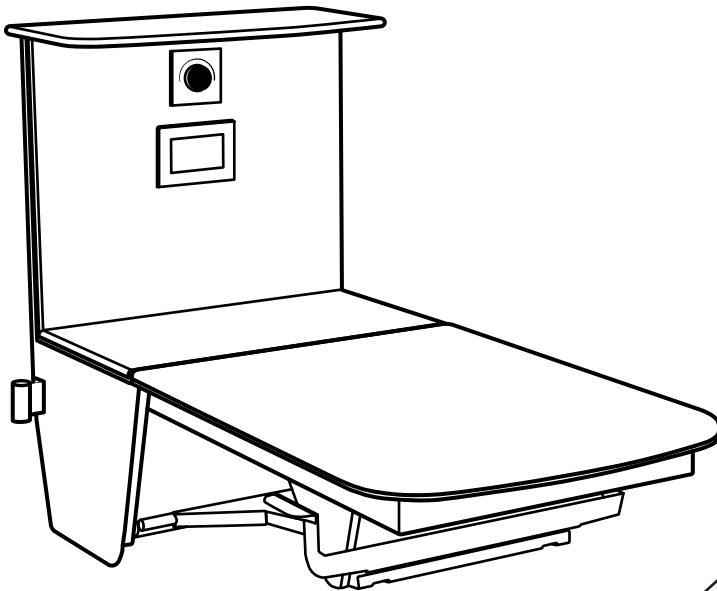

Base Unit

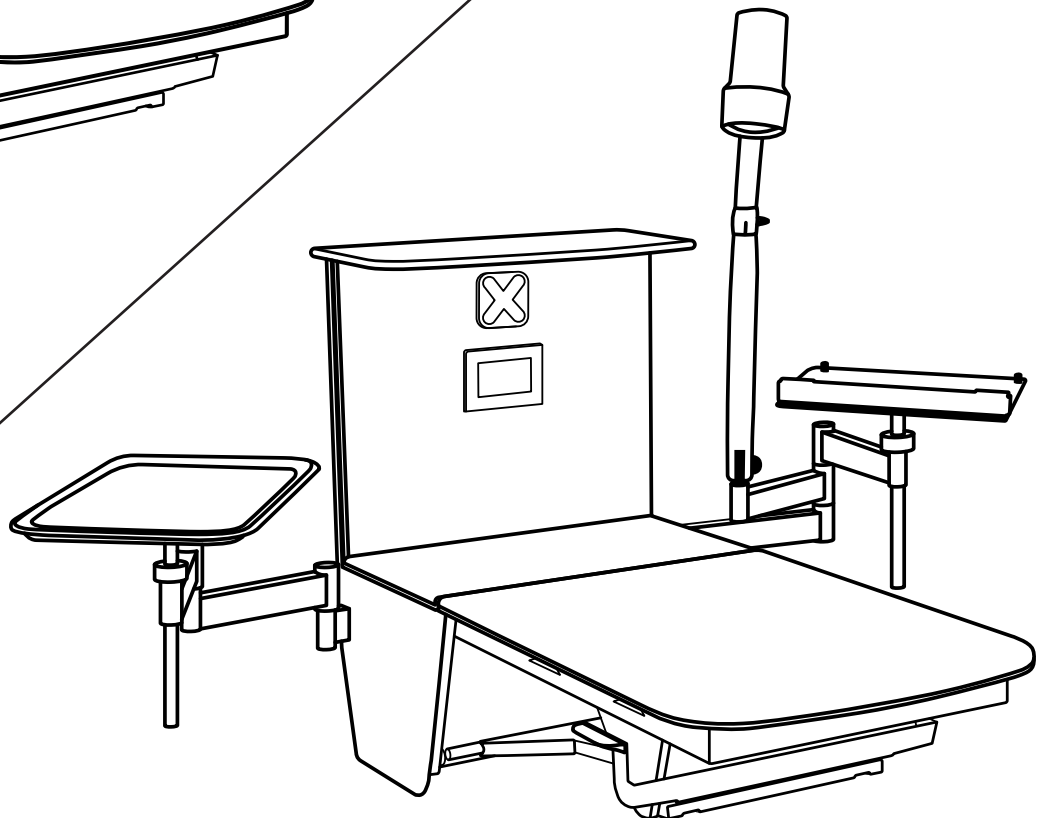

With Optional Accessories

About this Manual

This manual contains information on the setup, use and upkeep of the Olympic Elite Folding Exam Station™. It should be read in its entirety before first use to assure safe operation and optimal performance.

This Olympic Elite Folding Exam Station has a serial number you will need to reference when seeking technical support. This serial number as well as safety information can be found on the left side of the table behind the backslash. For future reference, it is recommended you write this serial number on the front of this manual.

Table of Contents

Component Identification.....	3
Introduction	4
Safety	4
Unpacking and Setup	5
Assembly.....	5
Operation	5–6
Raising and Lowering the Folding Tabletop	5
Precision Scale	5
Swing Arms.....	5
LED Sterility Indicator Lights	6
Storage Shelf	6
Corian Table Surface.....	6
Tie-Down Cleats	6
Power	6
Onboard Power Strips with USB ports	6
Cleaning	7
LED Sterility Indicator System	7
Maintenance.....	7
Specifications.....	7
Ordering Information	7
Optional Accessories.....	7
Summary of Precautions	8
Warranty.....	9
Contact Us	9

List of Figures

Figure 1: Component Identification.....	3
Figure 2: Unpacking Instructions	5
Figure 3: LED Lighting Control Panel.....	6

Component Identification

The following illustration demonstrates the key components of the Olympic Elite Folding Exam Station.

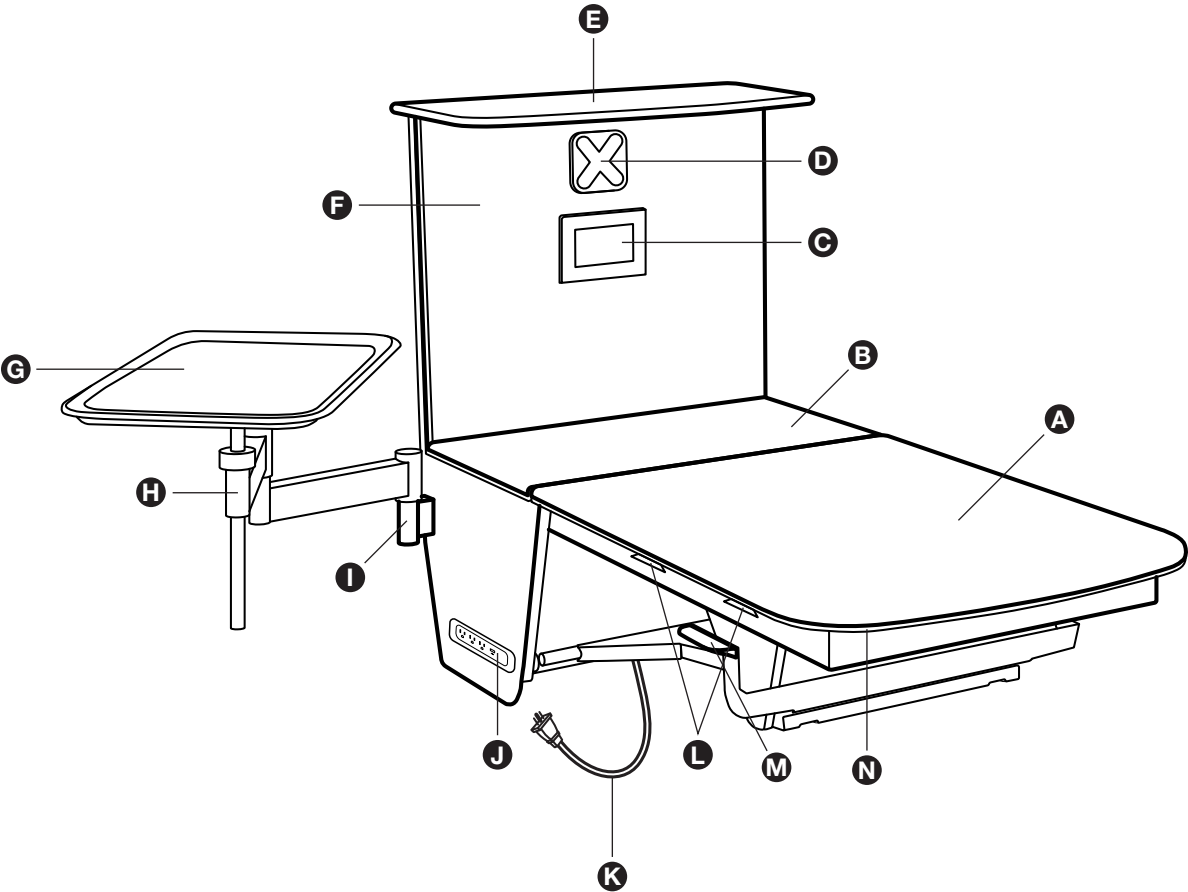


Figure 1

Item	Description		
A	Weighing Platform (Folds Down)	H	Olympic Standard Swing Arm (Optional)
B	Fixed Platform	I	Swing arm socket (each side)
C	Scale Display	J	Power Strip with USB-A and -C (each side)
D	Lighting Control Pad	K	10' Power Cord
E	Storage Shelf	L	Tie Down Cleats (2 each side)
F	Backsplash	M	Tabletop Release Handle (each side)
G	Height-Adjustable Instrument Tray (Optional)	N	LED Sterility Indicator Light Strip

Introduction

The Olympic Elite Folding Exam Station is a wall-mounted, foldable exam table for the modern exam room that includes a built-in precision scale, sterility indicator lights, two integrated power strips and a dedicated storage shelf. It also includes two sockets that can accommodate Olympic swing arms, which allow the user to customize the table by integrating all the ancillary equipment typically required in an exam room.

The Folding Exam Station features Olympic's Soft-Touch Release Handles. For safety, these contoured handles can only be activated after a patient is removed from the tabletop. Pressing up on either handle will cause the tabletop to fold down in a controlled manner, coming to rest quietly. The table is designed to accommodate patients up to 100 pounds.

The Folding Exam Station features Olympic's LED Sterility Indicator lighting system, a controllable LED light band that gently lights the perimeter of the unit. The LED Lighting Control Panel allows the user to change the status from Red (indicating that the table is waiting to be cleaned) to Green (Ready for use). The Control Panel also features Blue and Pink LED light settings which can be used to indicate a fractious patient, a private-pay patient, or any other message your hospital wishes to communicate.

Each optional Swing Arm includes two sockets that can accommodate any of Olympic's drop-in accessories such as a Height-Adjustable Instrument Tray, Laptop Tray, LED Exam Light, Hands-Free Magnifier with Wood's Lamp and more. The Folding Exam Station also features a storage shelf that can hold up to 20 pounds of supplies, placing everything you need at your fingertips.

Safety

Please read and follow the instructions in this manual, including all safety instructions and precautions, carefully and completely. The following safety notices are used throughout this manual:

Equipment Alert: Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Caution: Indicates a potentially hazardous situation which could result in minor or moderate personal injury or minor damage to the equipment if not avoided.

! WARNING !

Indicates a hazardous condition which could result in serious injury or major damage to the equipment if not avoided.

The user should be aware that the Olympic Elite Folding Exam Station is a powerful machine with moving parts capable of causing injury if proper precautions are not taken. The folding platform is raised and lowered by means of a manual release handle and a locking gas strut. Care must be taken to assure that hands, feet and other body parts are not in the path of the folding mechanism.

Please read and follow the instructions in this manual, including all safety instructions and precautions, carefully and completely.

For a summary of all precautions, please see page 8.

Unpacking and Setup

If you suspect concealed shipping damage, please save all packaging and contact Olympic Veterinary immediately at 1-800-789-6867 or send photos and description to service@olyvet.com.

1. Move the pallet to a level location as close as possible to the intended installation site.
2. If your Folding Exam Station was ordered with accessories such as an Instrument Tray, LED light, etc., you will find the accessory box(es) strapped to the top of the Folding Exam Station box. Remove the accessory box(es) and set aside until installation of the Folding Exam Station has been completed.
3. Remove the strap and lift off the cardboard lid. Remove the cardboard shroud.
4. Remove the two protective cardboard covers and foam blocking.

Equipment Alert: Do not grab the table by the Soft-Touch Release Handles.

5. Locate the included Mounting Hanger, Lag Screws, Washers and paper template and set aside. Follow the Olympic Elite Folding Exam Station Installation Instructions.

Assembly

The Olympic Elite Folding Exam Station arrives fully assembled.

Operation

Raising and Lowering the Folding Tabletop

There are two Tabletop Release Handles **M** (Fig. 1), one on each side of the table. Pressing on either handle will cause the tabletop to fold down gently.

1. To lower the tabletop, lift up on either handle. The locking gas strut will engage and the tabletop will smoothly descend on its own to its lowered position.
2. To raise the tabletop, simply lift from the outside edge of the folding platform until it locks in place.

Caution

- Be sure to keep all operator and patient body parts clear of the folding mechanism while raising or lowering the tabletop.
- Do not load patients exceeding 100 lbs on the Elite Folding Exam Station.

Equipment Alert: Assure that swing arms and Olympic accessories are clear of the folding section of the Folding Exam Station when lowering or raising the platform.

Precision Scale

The Folding Exam Station features an integrated, precision scale. No calibration required. Accuracy is ± 0.1 pounds and can weigh patients up to 100 lbs. See Olympic's Precision Scale Operator's Manual for use and calibration instructions.

Swing Arms

The Olympic Folding Exam Station features two sockets **I** (Fig. 1), one on each side of the station, that hold Olympic Swing Arms **H** (Fig. 1)

Each Swing Arm set can accommodate Olympic's suite of drop-in accessories. The tool-free design makes it easy to switch out components throughout the day to meet changing examination needs. For a list of all available Olympic accessories, see "Ordering Information" on page 7.

Equipment Alert:

- Assure the maximum weight of all accessories installed on each swing arm does not exceed 30 lbs.
- Assure that swing arms are clear of the folding section of the Elite Folding Exam Station when lowering or raising the platform.

Storage Shelf


The 22" x 11" storage shelf provides all the space you need for up to 20 lbs of supplies or equipment.

Equipment Alert: Assure the maximum weight placed on upper storage shelf does not exceed 20 lbs.

LED Sterility Indicator Lights

The Lighting Control Pad (*Figure 3*) provides control over the color and brightness of the Olympic LED Lighting System.

1. Press the On/Off Button **A** to turn system on or off.
2. Adjust brightness up or down by pressing and holding **B** or **C**.
3. Select LED color by pressing:
 - D** Red - Indicates table is waiting to be cleaned
 - E** Green - Indicates table is ready for next patient
 - F** Blue - To be assigned by hospital
 - G** Pink - To be assigned by hospital

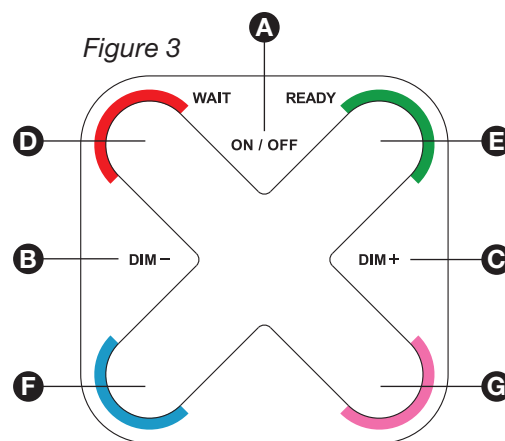

Item	Description
A	Turns On/Off LED lighting
B	Decrease LED brightness
C	Increase LED brightness
D	Set LEDs to Red
E	Set LEDs to Green
F	Set LEDs to Blue
G	Set LEDs to Pink

Corian® Table Surface

The working surfaces of the Elite Folding Exam Station are clad in durable Corian®.

Tie-Down Cleats

Should a patient require additional restraint, four integrated tie-down cleats **L** (*Fig. 1*) can be found on the sides of the folding tabletop.

Power

The Olympic Folding Exam Station operates on 120V/60Hz line voltage and plugs into any standard wall outlet. The Folding Exam Station features a 10' power cord **K** (*Fig. 1*) that supports all of the electrical needs of the Elite Folding Exam Station.

Onboard Power and USB Strips

Two onboard power strips **J** (*Fig. 1*), each with 3 electric outlets, one USB-A and one USB-C port, are located on the bottom right and bottom left sides of the Folding Exam Station.

Cleaning

The Folding Exam Station should be disinfected before each exam using a soft cloth with water or a mild disinfectant. Care should be taken to minimize the amount of cleaning or other fluids used on the scale display, light controls and around the onboard power and USB strips.

Properly maintained, the powder coated base and Corian® surface on your table should look good even after many years of service. While minor scuffing and scratching is to be expected over time, it can be minimized by avoiding the use of abrasive cleaning products or abrasive applicators such as "steel wool" or hard plastic scrubbers.

Equipment Alert: Powder coated surfaces should be cleaned only with a soap solution or alcohol-based cleaner. The use of bleach or a hydrogen peroxide solution such as Rescue can permanently damage the powder coated surface. Any damage caused by use of such solutions voids the product warranty on the product finish.

LED Sterility Indicator System

It is recommended to use the Red and Green buttons on the Lighting Control Pad to visually communicate the cleanliness/readiness of the exam room. After an examination is performed, press the “Wait” button on the Lighting Control Pad to turn the LED lights Red, indicating to hospital staff that the room has not been cleaned and is not yet ready for use. After the Folding Exam Station has been cleaned, press the “Ready” button on the Lighting Control Pad to turn the LED lights Green, indicating that the room is clean and ready for the next patient.

Equipment Alert: Do Not spray cleaning fluids directly onto the scale and lighting controls and the power outlets. Wipe with a moist cloth to clean.

Maintenance

The Olympic Elite Folding Exam Station is designed to provide many years of trouble-free performance with minimal servicing. The folding mechanism does not require any routine maintenance. Olympic Veterinary recommends a thorough cleaning and inspection annually.

For questions about the use or maintenance of the Olympic Elite Folding Exam Station, please call 1-800-789-6768.

Specifications

Olympic Elite Folding Exam Station™ – Model 52001, 52002

Patient Capacity: 100 lbs with centered load

Storage Shelf Capacity: 20 lbs

Swing Arm Capacity: 30 lbs

Table size: 22"W x 40.25"L

Overall maximum dimensions: 22"W x 43.5"L x 31.75"H

Power: 120V/60Hz

Weight: 130 lbs

Ordering Information

52001 – Olympic Elite Folding Exam Station™ with Corian® tabletop, backsplash and storage surfaces. Built-in Scale, LED sterility indicator lights, selectable LED Lighting, 2 three-socket power strips with USB-C and -A outlets, Storage Shelf, and 2 sockets ready for adding Olympic accessories.

52002 – Olympic Elite Folding Exam Station™ with Corian® tabletop, backsplash and storage surfaces. Built-in Scale, LED sterility indicator lights, selectable LED Lighting, 2 three-socket power strips with USB-C and -A outlets and Storage Shelf. Includes 2 Olympic Swing Arms with 2 sockets each, Swiveling Laptop Tray, Standard Height-Adjustable Instrument Tray and an Olympic LED Exam Light.

Optional Accessories

51086 – Olympic Swing Arm with 2 sockets for accessories

50996 – 10" Swing Arm Extension

51087 – Instrument Tray 13" X 10", Height-adjustable

51884 – Instrument Tray 19" X 13", Height-adjustable

51081 – IV Pole

50988 – Laptop Tray

50990 – Patient Monitor Tray

50970 – Olympic LED Exam Light

50971 – Olympic Advanced LED Procedure Light

50972 – Olympic Solo-Max LED Procedure Light

50973 – Olympic Hands-Free Magnifier with Wood's Light

Summary of Precautions

Equipment Alerts:

- Assure that swing arms and Olympic accessories are clear of the folding section of the Elite Folding Exam Station when lowering or raising the platform.
- Assure that swing arms are clear of the folding section of the Elite Folding Exam Station when lowering or raising the platform.
- Assure the maximum weight of all accessories installed on each swing arm does not exceed 30 lbs.
- Assure that maximum weight placed on upper storage shelf does not exceed 20 lbs.
- **Do Not** spray cleaning fluids directly onto the scale and lighting controls and the power outlets. Wipe with a moist cloth to clean.
- Powder coated surfaces should be cleaned only with a soap solution or alcohol-based cleaner. The use of bleach or a hydrogen peroxide solution such as Rescue can permanently damage the powder coated surface. Any damage caused by use of such solutions voids the product warranty on the product finish.

Cautions:

- Do not load patients exceeding 100 lbs on the Elite Folding Exam Station.
- Be sure to keep all operator and patient body parts clear of the folding mechanism while raising or lowering the tabletop.

! WARNING !

- The Elite Folding Exam Station is a heavy piece of equipment that should only be moved by a freight professional with the assistance of a dolly or other wheeled device.
- Maximum load per swing arm is 30 lbs.

3-Year Warranty

The Olympic Elite Folding Exam Station™ (Model 52001, 52002) is warranted for a period of three (3) years from the date of invoice.

Olympic Veterinary will elect to repair or replace a defective component without charge within the warranty period. In some cases, at Olympic Veterinary's sole discretion, it may be possible to provide the user with a repair kit, authorizing the user's technical staff to complete the repair.

This warranty is not valid for damage resulting from negligence, accident, unreasonable use, abuse or misuse, unauthorized modification or failure to comply with all Cautions, Warnings and Equipment Alerts noted in the product's Operator's Manual and Installation Guide.

Common wear parts such as casters, batteries, stretcher covers, etc. are warranted for a period of one (1) year from date of invoice.

Olympic Veterinary expressly disclaims and excludes any liability for consequential, incidental, indirect or punitive damages or financial loss to people or property resulting from any breach of warranty or from the improper operation of this equipment.

This limited warranty is provided to the original purchaser and is not transferable. This constitutes the complete and final agreement for Olympic Veterinary's Olympic Elite Folding Exam Station™ (Model 52001, 52002).

Contact Us

Olympic Veterinary Corporation, Customer Service

16040 Christensen Road, Suite 205
Tukwila, WA 98188

USA Phone: 1-800-789-6867

International Phone: +1 (206) 722-3777

Email: Service@OlyVet.com

NOTE: THIS IS NOT THE STREET ADDRESS TO BE USED FOR RMA RETURNS.

Detailed shipping instructions will be provided by customer service when an RMA is issued.